# EDGEBIOCERAMIC™ SEALER

en	INSTRUCTIONS FOR USE.....	2
cs	NÁVOD K POUŽITÍ.....	6
da	BRUGSANVISNING .....	10
de	GEBRAUCHSANWEISUNG.....	14
es	INSTRUCCIONES DE USO .....	19
fr	INSTRUCTIONS D’UTILISATION .....	23
it	ISTRUZIONI PER L’USO .....	28
nl	GEBRUIKSAANWIJZING .....	32
no	BRUKSANVISNING .....	36
pl	INSTRUKCJA UŻYCIA.....	40
pt	INSTRUÇÕES DE UTILIZAÇÃO.....	45
sk	NÁVOD NA POUŽITIE .....	49
sv	BRUKSANVISNING .....	53
tr	KULLANMA TALIMATI .....	57

US Endodontics, LLC  
2809 West Walnut Street  
Johnson City, TN 37604  
USA  
1-855-985-3636  
www.EdgeEndo.com

Innovative BioCeramix Inc.  
101 – 8218 North Fraser Way  
Burnaby, BC V3N 0E9  
Canada

Emergo Europe  
Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands

en

## INSTRUCTIONS FOR USE

### Contraindications:

- Do not use EDGEBIOCERAMIC SEALER in patients with a known allergy to any of the product's ingredients. An allergic reaction may require retreatment.

### Precautions:

Do not use after the expiration date


Do not re-use the disposable syringe tips (Tips)

- Discard the Tips after each application. Potential cross contamination may occur if single use syringe tips are reused or not properly cleaned before use.
- EDGEBIOCERAMIC SEALER and Tips are not designed to be sterilized. Failure to follow these instructions could distort the tips and/or damage the product resulting in procedural delays or user inconvenience.

### Cleaning:

- a. Disinfect the exterior surfaces of the syringe and syringe cap (once it is tightly sealed onto the syringe) prior to storage to reduce the risk of cross- contamination.
  - b. The EDGEBIOCERAMIC SEALER syringe should be mantled with a hygienic single use barrier sleeve for infection control for direct intra-oral use.
- Ensure there is continuous flow of material and the placement site is completely filled. Failure to do so may result in procedural delays.
  - Ensure that any bleeding is under control prior to placing the EDGEBIOCERAMIC SEALER as the material may wash out of the placement site and require retreatment.

### Storage:

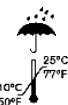Keep Dry

Store at room temperature

- Closely follow the recommended storage conditions. Failure to do so will cause the material to prematurely set resulting in retreatment of material placement or user inconvenience. To avoid prematurely inducing the setting process closely follow these guidelines:
  - a. Use the syringe cap to keep the syringe tightly closed when the material is not in use. Keep the syringe cap free from moisture.
  - b. Keep EDGEBIOCERAMIC SEALER tightly sealed in its pouch and store at room temperature in a dry area to avoid moisture contact.

### Warnings:

Irritant

Do not use if package is damaged

- Use personal protective equipment to avoid contact of EDGEBIOCERAMIC SEALER with the skin, mucus membranes and eyes. Unset material may cause irritation.
- Do not use excessive force to apply the material into the root canal as this may cause patient sensitivity/discomfort or breakage of the syringe plunger.

- Always inspect the syringe prior to application into the site. Using a syringe with illegible reference markings could lead to overfilling or underfilling of the root canal site.
- EDGEBIOCERAMIC SEALER has not been tested on pregnant or nursing mothers.
- Procedural delays or user inconvenience may be experienced if the Tip is not inspected prior to use. If the material does not flow out of the syringe tip or if the syringe tip feels stiff, discard the tip and use a new one.
- Always check the expiration date of the product to prevent procedural delays or user inconvenience (e.g. material becomes brittle or will not set).
- Overfilling the root canal may lead to patient sensitivity, foreign body inflammation, maxillary sinus aspergillosis, paresthesia of anesthesia due to nerve impingement or may require surgical removal of the overfill.
- Carefully read package labeling to ensure use of the appropriate bioceramic material. Failure to do so may cause user or patient inconvenience.
- Multiple continuous applications of material using the syringe delivery system may cause hand fatigue.

### **Product Description:**

EDGEBIOCERAMIC SEALER is a premixed ready-to-use injectable bioceramic paste developed for permanent root canal filling and sealing applications. EDGEBIOCERAMIC SEALER is an insoluble, radiopaque and aluminum-free material based on a calcium silicate composition, which requires the presence of water to set and harden. EDGEBIOCERAMIC SEALER does not shrink during setting and demonstrates excellent physical properties. EDGEBIOCERAMIC SEALER is packaged in a pre-loaded syringe and is supplied with disposable Tips. EDGEBIOCERAMIC SEALER may be delivered into the canal via the tips provided or it can be delivered via traditional methods.

### **Indications For Use:**

- Permanent obturation of the root canal following vital pulp-extirpation.
- Permanent obturation of the root canal following removal of infected or necrotic pulp and placement of intracanal dressings.

EDGEBIOCERAMIC SEALER is suitable for use in the single cone and lateral condensation technique.

### **Working Time:**

No mixing is required. EDGEBIOCERAMIC SEALER can be applied immediately and introduced directly into the root canal. The working time can be more than 30 minutes at room temperature.

### **Setting Time:**

Setting time is 4 hours. However, in very dry root canals, the setting time can be more than 10 hours.

### **Interactions:**

The setting time of EDGEBIOCERAMIC SEALER is dependent upon the presence of moisture in the dentin. The amount of moisture necessary to complete the setting reaction is naturally present within the dentin. Therefore, it is not necessary to add moisture in the root canal prior to placing the material.

### **Removal of Root Canal Filling:**

Conventional techniques can be used for the removal of EDGEBIOCERAMIC SEALER as used in combination with Gutta-Percha Points. Piezo Electric Ultrasonics (with water spray) can also be used.

## Directions For Use:

1. Prior to the application of EDGEBIOCERAMIC SEALER, thoroughly prepare and irrigate the root canal using standard endodontic techniques.
2. Dry the canal as you normally would using paper points.
3. Sealer Delivery Methods: both of the below sealer delivery methods are effective. The clinician should utilize whichever method they are most comfortable with.
  - a. Traditional Method:  
Coat the primary gutta percha point or a small hand file with sealer (extra-orally) and use the point or file to apply the sealer to the canal walls.
  - b. Tip Delivery Method:  
Remove the syringe cap from the EDGEBIOCERAMIC SEALER syringe. Securely attach a Tip with a clockwise twist to the hub of the syringe. Tips are flexible and can be bent to facilitate access to the root canal. Insert the tip of the syringe into the canal no deeper than the coronal one third (1/3). Gently and smoothly dispense a small amount (1-2 reference markings) of EDGEBIOCERAMIC SEALER into the root canal by compressing the plunger of the syringe. Using a #15 handfile or any comparable handfile, lightly coat the canal walls with the existing sealer in the canal. Then coat the master gutta percha cone with a thin layer of sealer and very slowly insert it into the canal. The master gutta percha cone will carry sufficient sealer to the apex.

**Note:** The precise fit of the Gutta Percha master cone creates excellent hydraulics and for that reason it is recommended that the practitioner use a small amount of sealer. Furthermore, it is also important to insert the master cone very slowly to its final working length. If using a warm vertical technique, see the instructions for use for your powered heat source.

4. If desired, place additional gutta percha points into the canal using standard condensation techniques.
5. Using a heat source, burn off the gutta percha cones at the orifice, apply a slight amount of vertical condensation, and remove any excess sealer with a moist cotton ball or swab.
6. EDGEBIOCERAMIC SEALER will remain part of the permanent root canal filling.
7. After each application, remove the Tip from the syringe with a counterclockwise twist to the hub of the syringe and discard. Clean the outside of the syringe and remove any excess paste, place the syringe cap tightly onto the syringe hub, and place the syringe into the foil pouch and ensure to seal the pouch. Store the pouch in a dry area at room temperature.

U.S. Patent Nos. 7,553,362, 7,575,628, 8,343,271, 8,475,811  
 European Patent Nos. 1861341 A4, 2142225 B1

Symbol	Meaning	Reference Number
	Catalogue Number	ISO 15223-1
	Batch Code	ISO 15223-1
	Use-by date	ISO 15223-1
	Quantity in the package	N/A
	Do not use if package is damaged	ISO 15223-1
	Keep Dry	ISO 15223-1
	Temperature Limit	ISO 15223-1
	Consult instructions for use; consult electronic instructions for use on the website listed (eIFU)	ISO 15223-1
	Irritant	GHS
	Do not re-use	ISO 15223-1
	Non-sterile	ISO 15223-1
	Medical Device	ISO 15223-1
	Unique device identifier	ISO 15223-1
	Authorized representative in the European Community/European Union	ISO 15223-1
	Manufacturer	ISO 15223-1
	Distributor	ISO 15223-1
	Conformité Européene. EU mandatory conformity marking	(EU) 2017/745 Annex V and 93/42/ EC Annex XII

US Endodontics, LLC  
 2809 West Walnut Street  
 Johnson City, TN 37604  
 USA  
 1-855-985-3636  
[www.EdgeEndo.com](http://www.EdgeEndo.com)

Innovative BioCeramix Inc.  
 101 – 8218 North Fraser Way  
 Burnaby, BC V3N 0E9  
 Canada

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0413

## 1 – IDENTIFICATION

Product Identifier	Trade Name	Product Identifier Number (REF)
	EDGEBIOCERAMIC™ SEALER 2g	USEBIOCSEU
Product Use	Root canal filling material.	
Restrictions on Use	For dental use only.	
Supplier	Innovative BioCeramix Inc. 101 - 8218 North Fraser Way Burnaby, BC V3N 0E9 Canada Tel: +1 604-221-6800	
Emergency Contact	For Chemical Emergency ONLY (spill, leak, fire, exposure, or accident) call: <b>CHEMTREC</b> Tel: <b>1-800-262-8200 (USA)</b> <b>+1 703-741-5500 (International)</b>	

## 2 – HAZARDS IDENTIFICATION

## GHS Classification and WHMIS 2015

Health Hazard	Hazard Class	Hazard Category	Signal Word	Hazard Statement	Pictogram
	Skin irritation	2	Warning	H315 Causes skin irritation	
	Eye irritation	2A	Warning	H319 Causes serious eye irritation	

Precautionary Statements	P	W
	P264	Wash hands thoroughly after handling.
	P280	Wear protective gloves, protective clothing, eye protection and face protection.
	P302 + P352	IF ON SKIN: Wash with plenty of soap and water.
	P321	Specific treatment: Wash skin thoroughly with cool water and pH-neutral soap or a mild detergent intended for use on skin.
	P332 + P313	If skin irritation occurs: Get medical advice / attention.
	P337 + P313	If eye irritation persists: Get medical advice / attention.
	P362 + P364	Take off contaminated clothing and wash it before reuse.
	P305 + P351 + P338	IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Physical Hazard	This product reacts chemically with water or moisture, and specific intermediate products of this reaction (i.e. calcium hydroxide, is present only while the product is setting or curing) can produce a strong alkaline material than that of the dry paste.
-----------------	--

Environment Hazard	Not applicable
--------------------	----------------

## 3 – COMPOSITION / INFORMATION ON INGREDIENTS

CAS #	EC Number	Index No.	REACH Registration No.	Concentration (wt%)	Chemical Name	Classification According to 67/548/EEC	Classification According to Regulation (EC) No. 1272/2008 (CLP)
1314-23-4	215-227-2	Not applicable	Not applicable	35.0 – 45.0%	Zirconium oxide	Irritant: Xi; R36 R38	Skin Irrit. 2 H315
12168-85-3	Not applicable	Not applicable	Not applicable	20.0 – 35.0%	Tricalcium silicate		Eye Irrit. 2 H319
10034-77-2	Not applicable	Not applicable	Not applicable	7.0 – 15.0%	Dicalcium silicate		
1305-62-0	215-137-3	Not applicable	Not applicable	1.0 – 4.0%	Calcium hydroxide		
Not applicable	Additives: Fillers: • None are classified as hazardous to the health or environment and do not contribute to the classification of the product. • The specific chemical identity and the exact concentration of the fillers in the product have been withheld as per Confidential Business Information (CBI).				Not applicable	Not applicable	Proprietary

## 4 – FIRST AID MEASURES

Route of Exposure	Most Important Symptoms/Effects, Acute and Delayed	First Aid Treatment
Skin Contact	<ul style="list-style-type: none"> <li>Skin irritation, drying, cracking or fissuring of the skin.</li> <li>Prolonged exposure can cause severe skin damage in the form of chemical (caustic) burns.</li> </ul>	<ul style="list-style-type: none"> <li>Wash skin thoroughly with cool water and pH-neutral soap or a mild detergent intended for use on skin.</li> <li>Wear gloves to remove any contaminated clothes and shoes from the individual.</li> <li>Get medical attention in all cases of prolonged exposure to the product, freshly cured product or prolonged wet skin exposure to the product.</li> </ul>
Eye Contact	<ul style="list-style-type: none"> <li>Severe irritation and/or inflammation.</li> <li>Blindness in severe cases.</li> </ul>	<ul style="list-style-type: none"> <li>Immediately rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do.</li> <li>Continue rinsing, include under lids to remove all particles.</li> <li>Get immediate medical attention.</li> </ul>
Inhalation	<ul style="list-style-type: none"> <li>Irritation to the moist mucus membranes of the nose, throat and upper respiratory system.</li> <li>Pre-existing upper respiratory and lung diseases may be aggravated.</li> </ul>	<ul style="list-style-type: none"> <li>Move exposed individual from the area into fresh air.</li> <li>Get medical attention if coughing and other symptoms do not subside.</li> </ul>
Ingestion	<ul style="list-style-type: none"> <li>Internal discomfort or irritation to the digestive tract.</li> </ul>	<ul style="list-style-type: none"> <li>Do NOT induce vomiting.</li> <li>If individual is conscious, use water to rinse out the mouth and have the individual drink plenty of water.</li> <li>If vomiting occurs, lean patient forward or place on left side (head-down position, if possible) to maintain an open airway and prevent aspiration. Observe the individual carefully.</li> <li>Never give anything by mouth to an unconscious individual (i.e. showing signs of being sleepy or with reduced awareness).</li> <li>Get immediate medical attention.</li> </ul>

## 5 – FIRE-FIGHTING MEASURES

Conditions of Flammability	Not flammable or combustible.
Suitable Extinguishing Media	Use water spray, alcohol-resistant foam, dry chemical or carbon dioxide.
Unsuitable Extinguishing Media	There is no restriction on the type of extinguisher.
Specific Hazards Arising From the Product	None known.
Special Protective Actions Equipment and Precautions for Fire-Fighters	<ul style="list-style-type: none"> <li>Wear self-contained breathing apparatus and protective gloves for firefighting.</li> <li>Equipment should be thoroughly decontaminated after use.</li> <li>Prevent spillage from entering drains or water courses.</li> </ul>

## 6 – ACCIDENTAL RELEASE MEASURES

Personal Precautions, Protective Equipment and Emergency Procedures	<ul style="list-style-type: none"> <li>Wear appropriate personal protective equipment to prevent contamination of skin, eyes and personal clothing (see Section 8).</li> <li>Avoid contact with the product: <ul style="list-style-type: none"> <li>Avoid contact to the product, freshly cured product or prolonged wet skin exposure to the product.</li> <li>Avoid actions that cause dried product to become airborne.</li> <li>Avoid inhalation of dried product dust.</li> </ul> </li> <li>Use a scoop or moist paper towel(s) to clean up spilled product.</li> </ul>
Environmental Precautions	<ul style="list-style-type: none"> <li>Prevent further spillage.</li> <li>Keep product away from drains, surface and ground water.</li> </ul>
Methods and Materials for Containment and Cleaning Up	<ul style="list-style-type: none"> <li>Use a scoop or moist paper towel(s) to clean up spilled product. Dispose of in a suitable sealed container.</li> <li>Dispose of waste product according to local state and federal regulations.</li> <li>Cover drains.</li> <li>Wash spilled area with large amounts of water and dry thoroughly.</li> </ul>

## 7 – HANDLING AND STORAGE

Precautions for Safe Handling	<ul style="list-style-type: none"> <li>Avoid contact with skin and eyes.</li> <li>Avoid inhalation of dried product.</li> <li>Wear personal protective equipment.</li> <li>Remove contaminated clothing and personal protective equipment before entering eating areas.</li> <li>Wash hands after handling product.</li> <li>Remove contaminated clothing and personal protective equipment. Wash before reuse.</li> <li>Avoid contact with ammonia, ammonia nitrate, chlorine or chlorine containing items.</li> </ul>
Conditions for Safe Storage	<ul style="list-style-type: none"> <li>Air and moisture sensitive.</li> <li>Store at room temperature in a dry area.</li> <li>Store tightly closed in the original packaging. Keep dry until used.</li> <li>Protect from excessive heat and cold.</li> <li>Protect packaging against physical damage.</li> <li>Store away from incompatible materials, and from foodstuffs and beverages.</li> <li>Do not store near ammonia, ammonia nitrate, chlorine or chlorine containing items.</li> </ul>

## 8 – EXPOSURE CONTROLS / PERSONAL PROTECTION

Control Parameters	The product does not contain any relevant quantities of materials with critical values that have to be monitored at the workplace.	
Appropriate Engineering Controls	Use and store in a well-ventilated area.	
Individual Protections Measures, Such as Personal Protective Equipment (PPE)	<b>Eye / Face Protection</b> 	<ul style="list-style-type: none"> <li>Safety glasses with side shields.</li> <li>Tightly sealed safety goggles.</li> <li>Face shields.</li> <li>Contact lenses should not be worn.</li> </ul>
	<b>Skin Protection</b> 	<ul style="list-style-type: none"> <li>Glove material has to be impermeable and resistant to the product: <ul style="list-style-type: none"> <li>Exam gloves: nitrile rubber or latex.</li> <li>Recommended thickness of the glove material: <math>\geq 0.38\text{mm}</math></li> </ul> </li> <li>Use proper glove removal technique (without touching glove's outer surface) to avoid skin contact with this product.</li> <li>Dispose of contaminated gloves after use in accordance with applicable laws and good laboratory practice.</li> <li>Wash and dry hands after handling product.</li> </ul>
		<ul style="list-style-type: none"> <li>Lightweight impermeable protective work clothing (lab coat), as appropriate, to prevent skin contact.</li> <li>Remove contaminated clothing and personal protective equipment before entering eating areas.</li> <li>Remove contaminated clothing and personal protective equipment. Wash before reuse.</li> </ul>
	<b>Respiratory Protection</b> 	<ul style="list-style-type: none"> <li>Surgical face mask</li> <li>Handle product in a well-ventilated area.</li> </ul>
	<b>Thermal Hazards</b>	Not applicable
<b>Hygiene Measures</b>	<ul style="list-style-type: none"> <li>Handle in accordance with good industrial hygiene and safety practice.</li> <li>Keep away from foodstuffs and beverages.</li> <li>Wash hands before breaks and at the end of work.</li> </ul>	

## 9 – PHYSICAL AND CHEMICAL PROPERTIES

Appearance	State	Solid (thick paste)	Upper / Lower Flammability or Explosive Limits	Not available
	Colour	White		
Odour	No distinct odour		Vapour Pressure	Not available
Odour Threshold	Not available		Vapour Density	Not available
pH	>12 (Basic)		Relative Density	Not available
Melting Point / Freezing Point	Not available		Solubility (with water)	Slightly soluble
Initial Boiling Point and Boiling Range	Not available		Partition Coefficient: n-octanol / water	Not available
Flash Point	Not available		Auto-ignition Temperature	Not available
Evaporation Rate	Not available		Decomposition Temperature	Not available
Flammability	Not available		Viscosity	Not available

## 10 – STABILITY AND REACTIVITY

<b>Reactivity</b>	Contact with water will result in hydration of the product and produces (caustic) calcium hydroxide.
<b>Chemical Stability</b>	<ul style="list-style-type: none"> <li>Stable under recommended storage conditions.</li> <li>Keep dry.</li> </ul>
<b>Possibility of Hazardous Reactions</b>	Not available
<b>Conditions To Avoid</b>	Avoid unintentional contact with water to prevent premature setting/curing; contact will result in hydration of the product and produces (caustic) calcium hydroxide.
<b>Incompatible Materials</b>	Incompatible with acids, ammonia, ammonia nitrate, ammonium salts, aluminum metal and chlorine.
<b>Hazardous Decomposition Products</b>	<ul style="list-style-type: none"> <li>Decomposition will not spontaneously occur.</li> <li>Contact with water will result in hydration of the product and produces (caustic) calcium hydroxide.</li> </ul>

## 11 – TOXICOLOGICAL INFORMATION

<b>Emergency Overview</b>	This product is a white paste that poses little immediate hazard. A single-term exposure to the paste is not likely to cause serious harm. However, exposure of sufficient amount and duration (either single or multiple) to the moist product can cause serious irritation to the skin and eyes in the form of chemical (caustic) burns. Similar tissue damage can occur if wet or moist areas of the body are exposed for sufficient duration to the product.
---------------------------	--

## Numerical Measures of Toxicity

Chemical Name	CAS #	ACGIH	OSHA PEL	LD <sub>50</sub>
Zirconium oxide	1314-23-4	5 mg/m <sup>3</sup> total dust	5 mg/m <sup>3</sup> TWA	Not available
Tricalcium silicate	12168-85-3	10 mg/m <sup>3</sup> total dust	5 mg/m <sup>3</sup> respiratory dust	Not available
Dicalcium silicate	10034-77-2		10 mg/m <sup>3</sup> total dust	
Calcium hydroxide	1305-62-0	5 mg/m <sup>3</sup>	5 mg/m <sup>3</sup> TWA (respirable fraction) 15 mg/m <sup>3</sup> TWA (total dust)	7300 mg/kg

Routes of Exposure	Symptoms Related to Physical, Chemical and Toxicological Characteristics	Delayed and Immediate Effects Chronic Effects from Short and Long-Term Exposure	Target Organ(s)
<b>Skin Contact</b>	Skin irritation, drying, cracking or fissuring of the skin.	Severe skin damage in the form of chemical (caustic) burns.	Skin
<b>Eye Contact</b>	Severe irritation and/or inflammation.	Blindness in severe cases.	Eye
<b>Inhalation</b>	Irritation to the moist mucus membranes of the nose, throat and upper respiratory system.	Pre-existing upper respiratory and lung diseases may be aggravated.	Mucous membranes of nose, throat and upper respiratory system.
<b>Ingestion</b>	Internal discomfort or irritation to the digestive tract.	Ill effects are possible if large quantities are consumed.	Digestive system

<b>Acute Toxicity</b>	Product is not classified as a toxic substance.
<b>Skin Corrosion / Irritation</b>	May cause skin irritation.
<b>Serious Eye Damage / Irritation</b>	May cause serious eye irritation.
<b>Respiratory or Skin Sensitization</b>	No allergic potential after multiple uses and good tolerance by subcutaneous tissues.
<b>Germ Cell Mutagenicity</b>	No mutagenic effect.
<b>Carcinogenicity (NTP, IARC, OSHA)</b>	Ingredients not listed as carcinogens.
<b>Reproductive Toxicity</b>	Not available
<b>Specific Target Organ Toxicity (STOT) – Single Exposure</b>	Product is not classified as a toxic substance.
<b>Specific Target Organ Toxicity (STOT) – Repeated Exposure</b>	Product is not classified as a toxic substance.
<b>Aspiration Hazard</b>	Not available
<b>Interactive Effects</b>	Not available
<b>Other Information</b>	Not applicable

## 12 – ECOLOGICAL INFORMATION

<b>Toxicity</b>	Not available	<b>Mobility in Soil</b>	Not available
<b>Persistence and Degradability</b>	Not available	<b>Other Adverse Effects</b>	No known significant effects or critical hazards.
<b>Bioaccumulative Potential</b>	Not available		

## 13 – DISPOSAL CONSIDERATIONS

<b>Disposal Methods</b>	<ul style="list-style-type: none"> <li>For the safe handling of disposing of this product, wear appropriate personal protective equipment to prevent contamination of skin, eyes and personal clothing (see Section 8).</li> <li>Dispose of waste product or contaminated product in a suitable sealed container.</li> <li>Dispose of product according to local state and federal regulations.</li> </ul>
<b>Physical / Chemical Properties Affecting Disposal Options</b>	No known precautions.
<b>Discourage Sewage Disposal</b>	Do not dispose of product down drains or from entering sewage systems.
<b>Special Precautions for Incineration or Landfill</b>	No known precautions.

## 14 – TRANSPORT INFORMATION

<b>UN Number</b>	Not applicable, the product is not classified as Dangerous Goods.
<b>UN Proper Shipping Name</b>	Not applicable, the product is not classified as Dangerous Goods.
<b>Transport Hazard Class(es)</b>	Not classified as a transport hazard in accordance with the UN Model Regulations.
<b>Packing Group</b>	Not applicable, the product is not classified as Dangerous Goods.
<b>Environmental Hazards</b>	<ul style="list-style-type: none"> <li>Product is not known as a marine pollutant according to the IMDG Code.</li> <li>Product is not an environmental hazard according to the UN Model Regulations, ADR, RID and ADN.</li> </ul>
<b>Precautions for User</b>	Not applicable
<b>Transport in Bulk (According to Annex II of MARPOL 73/78<sup>9</sup> and the IBC Code)</b>	Not applicable

## 15 – REGULATORY INFORMATION

<b>Status under REGULATION (EC) No 1272/2008 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 16 December 2008 on classification, labelling and packaging of substances and mixtures, amending and repealing Directives 67/548/EEC and 1999/45/EC, and amending Regulation (EC) No 1907/2006;</b>	<ul style="list-style-type: none"> <li>This device is classified as an irritant: <ul style="list-style-type: none"> <li>Skin irritation (Category 2)</li> <li>Eye irritation (Category 2)</li> </ul> </li> </ul>
<b>Status under USDOL-OSHA Hazard Communication Rule, 29 CFR 1910.1200:</b>	<ul style="list-style-type: none"> <li>The ingredients in this product (calcium silicate compounds, zirconium oxide and calcium hydroxide) are considered to be “hazardous chemicals” under this regulation.</li> </ul>

## 16 – OTHER INFORMATION

<b>Date of Preparation</b>	2023-08-15
----------------------------	------------

This product should be used according to manufacturer's directions. Safety precautions presented above may not be adequate for all individuals or situations. Inexperienced product users should obtain proper training before using or handling this product. Users have the responsibility to comply with all laws and procedures applicable to the products' safe handling and use, determine the suitability of the product for its intended use, and understand possible hazards associated with mixing this product with other materials. The above information is believed to be correct but does not purport to be all inclusive and shall be used only as a guide. The information in this document is based on the present state of our knowledge and is applicable to the product with regard to appropriate safety precautions. It does not represent any guarantee of the properties of the product. The supplier and/or manufacturer of this product shall not be held liable for damage resulting from handling, misuse or from contact with this product.