EDGEBIOCERAMIC RETROFILL™ AND PERFORATION REPAIR MATERIAL

en	INSTRUCTIONS FOR USE.....	2
cs	NÁVOD K POUŽITÍ.....	7
da	BRUGSANVISNING	12
de	GEBRAUCHSANWEISUNG	17
es	INSTRUCCIONES DE USO	23
fr	INSTRUCTIONS D’UTILISATION	29
it	ISTRUZIONI PER L’USO	35
nl	GEbruIKSAANWIJZING	41
no	BRUKSANVISNING	47
pl	INSTRUKCJA UŻYCIA.....	52
pt	INSTRUÇÕES DE UTILIZAÇÃO.....	58
sk	NÁVOD NA POUŽITIE	64
sv	BRUKSANVISNING	70
tr	KULLANMA TALIMATI	76

US Endodontics, LLC
2809 West Walnut Street
Johnson City, TN 37604
USA
1-855-985-3636
www.EdgeEndo.com

Innovative BioCeramix Inc.
101 – 8218 North Fraser Way
Burnaby, BC V3N 0E9
Canada

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

en

INSTRUCTIONS FOR USE

Contraindications:

- Do not use EDGEBIOCERAMIC RETROFILL in patients with a known allergy to any of the product's ingredients. An allergic reaction may require re-treatment.

Precautions:

Do not use after the expiration date

- EDGEBIOCERAMIC RETROFILL is not designed to be sterilized. Failure to follow these instructions could damage the product resulting in procedural delays or user inconvenience.
- Cleaning:
 - a. Disinfect the exterior surfaces of the syringe and syringe cap (once it is tightly sealed onto the syringe) prior to storage to reduce the risk of cross-contamination.
 - b. The EDGEBIOCERAMIC RETROFILL syringe should be mantled with a hygienic single-use barrier sleeve for infection control for direct intra-oral use.
- Ensure there is continuous flow of material and the placement site is completely filled. Failure to do so may result in procedural delays.
- Ensure that any bleeding is under control prior to placing the EDGEBIOCERAMIC RETROFILL as the material may wash out of the placement site and require re-treatment.

- Storage:

Keep Dry

Store at room temperature

- Closely follow the recommended storage conditions. Failure to do so will cause the material to prematurely set resulting in re-treatment of material placement or user inconvenience. To avoid prematurely inducing the setting process closely follow these guidelines:
 - a. Use the cap to keep the syringe tightly closed when the material is not in use. Keep the cap free of moisture.
 - b. Keep EDGEBIOCERAMIC RETROFILL tightly sealed in its pouch and store at room temperature in a dry area to avoid moisture contact.

Warnings:

Irritant

Do not use if package is damaged

- Use personal protective equipment to avoid contact of EDGEBIOCERAMIC RETROFILL with the skin, mucus membranes and eyes. Unset material may cause irritation.
- Do not use excessive force to apply the material into the root canal as this may cause patient sensitivity/discomfort or breakage of the syringe plunger.
- Always inspect the syringe prior to application into the site. Using a syringe with illegible reference markings could lead to overfilling or underfilling of the root canal site.
- EDGEBIOCERAMIC RETROFILL has not been tested on pregnant or nursing mothers.

- Always check the expiration date of the product to prevent procedural delays or user inconvenience (e.g. material becomes brittle or will not set).
- Overfilling the root canal may lead to patient sensitivity, foreign body inflammation, maxillary sinus aspergillosis, paresthesia of anesthesia due to nerve impingement or may require surgical removal of the overfill.
- Carefully read package labeling to ensure use of the appropriate bioceramic material. Failure to do so may cause user or patient inconvenience.
- Multiple continuous applications of material using the syringe delivery system may cause hand fatigue.

Product Description:

EDGEBIOCERAMIC RETROFILL is a ready-to-use, premixed bioceramic material developed for permanent root canal repair and surgical applications. It is an insoluble, radiopaque and aluminum-free material based on a calcium silicate composition, which requires the presence of water to set and harden. EDGEBIOCERAMIC RETROFILL does not shrink during setting and demonstrates excellent physical properties.

EDGEBIOCERAMIC RETROFILL is packaged in a preloaded syringe.

Indications For Use:

- Repair of Root Perforation
- Repair of Root Resorption
- Root End Filling
- Apexification
- Pulp Capping

Working Time:

No mixing is required. The setting reaction begins as soon as the material is placed in contact with a moist environment. The working time is more than 30 minutes.

Setting Time:

Setting time is a minimum of 2 hours in normal conditions, but can take longer to set in extremely dry canals.

Interactions:

The setting time of EDGEBIOCERAMIC RETROFILL is dependent upon the presence of moisture in the dentin. The amount of moisture necessary to complete the setting reaction is naturally present within the dentin. Therefore, it is not necessary to add moisture in the root canal prior to placing the material.

Directions For Use:

1. Prior to the application of EDGEBIOCERAMIC RETROFILL, thoroughly prepare and irrigate the root canal using standard endodontic techniques.
2. Unscrew the cap from the syringe.
3. When using EDGEBIOCERAMIC RETROFILL gently and smoothly extrude the desired amount of material from the syringe by compressing the plunger.

Note: Only a small amount of material is necessary to be removed from the syringe for each application.

4. Immediately after removing the material screw the cap tightly back on the syringe.
5. Use a sterile plastic instrument (of your choice) to place the material into the intended anatomic section of the root canal and compress the material with the plastic instrument.

6. Remove excess material with an appropriate sized spoon excavator or disposable microbrush.
7. Place the syringe into the foil pouch and be sure to seal the pouch. Store the pouch in a dry area at room temperature.

Indications For Use:

Pulp Capping:

Indirect

1. Indirect pulp caps have the best prognosis in cases of normal pulp or reversible pulpitis. Do not attempt an indirect pulp cap in cases of irreversible pulpitis.
2. Isolate the operative area with a rubber dam.
3. Prepare the cavity shape by removing any decay with a high-speed bur under a constant cooling water spray.
4. Before exposure occurs (0.5-1mm from the pulp), disinfect the internal surfaces of the cavity preparation and remove excessive moisture with a cotton pellet (do not air dry).
5. Place an adequate amount of EDGEBIOCERAMIC RETROFILL over the affected dentin near the pulp, extending onto normal dentin.
6. Remove excess with a spoon excavator or a micro brush.
7. Place a thin layer of glass ionomer cement over the repair material extending laterally onto clean dentin.
8. Once the glass ionomer is set, proceed to restore with a final restoration.

Direct

1. Once an exposure occurs, wash and disinfect the area thoroughly, control hemostasis, and prepare the exposure site for repair with EDGEBIOCERAMIC RETROFILL.
2. Place an adequate amount of the EDGEBIOCERAMIC RETROFILL over the perforation using a plastic instrument and remove excess with a curette and/or micro brush.
3. It is recommended to fill the entire cavity with a reinforced glass ionomer core material and observe the tooth for 4-6 weeks prior to final restoration with a composite material. The glass ionomer core can be used as a base during the subsequent visit.

Note: For deciduous teeth with substantial exposures, consider removing the pulp and following instructions 1-3 above.

Repair of Root Perforation:

1. Perforations have the best chance of success the sooner they are repaired. Repair the perforation as soon as it occurs or is noted.
 2. After isolation with a rubber dam, the area surrounding the perforation should be thoroughly and carefully cleaned and disinfected.
 3. Obtain adequate hemostasis from the perforation site and apply the EDGEBIOCERAMIC RETROFILL to the defect and seal all perforation margins.
 4. Remove any excess with a spoon excavator, or a micro brush, and create a flush perforation cavosurface.
 5. Take a radiograph to confirm an adequate seal. Add or remove EDGEBIOCERAMIC RETROFILL as needed.
- a. Single Visit Perforation Repair (small defects):

If you plan to complete root canal therapy during the same visit, apply a thin layer of self-cure or dual cure glass ionomer cement over the EDGEBIOCERAMIC RETROFILL and extend it onto sound dentin (cover the perforation material completely). Do not use composite material over the unset EDGEBIOCERAMIC RETROFILL as it will be difficult to create a bond. After the glass ionomer cap has set, complete the root canal procedure.

b. Two Visit Perforation Repair (large defects):

If the perforation area is too large and safe coverage of the EDGEBIOCERAMIC RETROFILL cannot be obtained with glass ionomer in a single visit (gently pushing the EDGEBIOCERAMIC RETROFILL through the defect) gently place a moist cotton pellet over the EDGEBIOCERAMIC RETROFILL and seal the access opening. Remove the cotton during the second visit and complete the root canal procedure.

Repair of Internal Root Resorption:

For Perforating Internal Root Resorption defects requiring sealing of the perforation see “Repair of Root Perforation” directions. For Non-Perforating Internal Root Resorption defects, consider simply obturating using EDGEBIOCERAMIC SEALER and Points.

Repair of External Root Resorption:

Subcrestal Defects

Remove all affected cementum and dentin until all resorptive cells are removed. Condition the root surface as desired (citric acid etch). Place EDGEBIOCERAMIC RETROFILL into the defect reestablishing the lost contours of the natural tooth and close the wound.

Supracrestal Defects

A glass ionomer compound is recommended in such cases.

Root End Filling:

1. Following apicoectomy and retropreparation clean and disinfect the retropreparation as usual.
2. Place an adequate amount of EDGEBIOCERAMIC RETROFILL into the retropreparation using a plastic instrument.
3. Condense or compress the material into the preparation from the bottom up to avoid trapping air until the preparation is completely sealed.
4. Remove any excess material using a micro-brush or a curette.
5. Radiograph the placement of the material to ensure its placement is adequate.
6. If placement is inadequate add or remove EDGEBIOCERAMIC RETROFILL as necessary.
7. Close the surgical opening after confirming that the root end preparation has been sufficiently sealed.
8. EDGEBIOCERAMIC RETROFILL is intended as a definitive repair material.

Apexification (Apical Barrier):

1. Isolate the operative area with a rubber dam.
2. Open and debride the root canal, irrigate thoroughly and dry the root canal.
3. If further disinfection is required, consider Calcium Hydroxide therapy for a week.
4. Place EDGEBIOCERAMIC RETROFILL into the apical area of the root until an apical plug of at least 3-5mm in depth is created.
5. Radiograph the placement of the material to ensure an adequate plug has been established.
6. Fill the remaining root canal space.
 - a. To fill with gutta percha during a subsequent visit, place a provisional in the access and revisit in a week to fill the remaining portion of the canal with EDGEBIOCERAMIC SEALER and Points.
7. Restore the access opening with your restorative material of choice.
8. EDGEBIOCERAMIC RETROFILL will remain as permanent part of the root canal apexification repair.

EDGEBIOCERAMIC RETROFILL™ AND PERFORATION REPAIR MATERIAL

U.S. Patent Nos. 7,553,362, 7,575,628, 8,343,271, 8,475,811
European Patent Nos. 1861341 A4, 2142225 B1

Symbol	Meaning	Reference Number
	Catalogue Number	ISO 15223-1
	Batch Code	ISO 15223-1
	Use-by date	ISO 15223-1
	Quantity in the package	N/A
	Do not use if package is damaged	ISO 15223-1
	Keep Dry	ISO 15223-1
	Temperature Limit	ISO 15223-1
	Consult instructions for use; consult electronic instructions for use on the website listed (eIFU)	ISO 15223-1
	Irritant	GHS
	Non-sterile	ISO 15223-1
	Medical Device	ISO 15223-1
	Unique device identifier	ISO 15223-1
	Authorized representative in the European Community/European Union	ISO 15223-1
	Manufacturer	ISO 15223-1
	Distributor	ISO 15223-1
	Conformité Européene. EU mandatory conformity marking	(EU) 2017/745 Annex V and 93/42/ EC Annex XII

US Endodontics, LLC
2809 West Walnut Street
Johnson City, TN 37604
USA
1-855-985-3636
www.EdgeEndo.com

Innovative BioCeramix Inc.
101 – 8218 North Fraser Way
Burnaby, BC V3N 0E9
Canada

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0413

1 – IDENTIFICATION

Product Identifier	Trade Name	Product Identifier Number (REF)
	EDGEBIOCERAMIC RETROFILL™ AND PERFORATION REPAIR MATERIAL 0.5G	USEBIORMEU
Product Use	Root canal repair and filling material.	
Restrictions on Use	For dental use only.	
Supplier	Innovative BioCeramix Inc. 101 - 8218 North Fraser Way Burnaby, BC V3N 0E9 Canada Tel: +1 604-221-6800	
Emergency Contact	For Chemical Emergency ONLY (spill, leak, fire, exposure, or accident) call: CHEMTREC Tel: 1-800-262-8200 (USA) +1 703-741-5500 (International)	

2 – HAZARDS IDENTIFICATION

GHS Classification and WHMIS 2015

Health Hazard	Hazard Class	Hazard Category	Signal Word	Hazard Statement	Pictogram
	Skin irritation	2		Warning	
Eye irritation	2A		Warning	H319 Causes serious eye irritation	

Precautionary Statements	P264	Wash hands thoroughly after handling.
	P280	Wear protective gloves, protective clothing, eye protection and face protection.
	P302 + P352	IF ON SKIN: Wash with plenty of soap and water.
	P321	Specific treatment: Wash skin thoroughly with cool water and pH-neutral soap or a mild detergent intended for use on skin.
	P332 + P313	If skin irritation occurs: Get medical advice / attention.
	P337 + P313	If eye irritation persists: Get medical advice / attention.
	P362 + P364	Take off contaminated clothing and wash it before reuse.
P305 + P351 + P338	IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.	

Physical Hazard	This product reacts chemically with water or moisture, and specific intermediate products of this reaction (i.e. calcium hydroxide, is present only while the product is setting or curing) can produce a strong alkaline material than that of the dry paste.
Environment Hazard	Not applicable

3 – COMPOSITION / INFORMATION ON INGREDIENTS

CAS #	EC Number	Index No.	REACH Registration No.	Concentration (wt%)	Chemical Name	Classification According to 67/548/EEC	Classification According to Regulation (EC) No. 1272/2008 (CLP)
12168-85-3	Not applicable	Not applicable	Not applicable	30.0 – 36.0%	Tricalcium silicate	Irritant: Xi; R36 R38	Skin Irrit. 2 H315
1314-23-4	215-227-2	Not applicable	Not applicable	15.0 – 18.0%	Zirconium oxide		
1314-61-0	215-238-2	Not applicable	Not applicable	12.0 – 15.0%	Tantalum pentoxide	Eye Irrit. 2 H319	
10034-77-2	Not applicable	Not applicable	Not applicable	9.0 – 13.0%	Dicalcium silicate		
10034-76-1	231-900-3	Not applicable	Not applicable	3.0 – 8.0%	Calcium sulfate		
Not applicable	Additives: Fillers: • None are classified as hazardous to the health or environment and do not contribute to the classification of the product. • The specific chemical identity and the exact concentration of the fillers in the product have been withheld as per Confidential Business Information (CBI).				Not applicable	Not applicable	Proprietary

4 – FIRST AID MEASURES

Route of Exposure	Most Important Symptoms/Effects, Acute and Delayed	First Aid Treatment
Skin Contact	<ul style="list-style-type: none"> Skin irritation, drying, cracking or fissuring of the skin. Prolonged exposure can cause severe skin damage in the form of chemical (caustic) burns. 	<ul style="list-style-type: none"> Wash skin thoroughly with cool water and pH-neutral soap or a mild detergent intended for use on skin. Wear gloves to remove any contaminated clothes and shoes from the individual. Get medical attention in all cases of prolonged exposure to the product, freshly cured product or prolonged wet skin exposure to the product.
Eye Contact	<ul style="list-style-type: none"> Severe irritation and/or inflammation. Blindness in severe cases. 	<ul style="list-style-type: none"> Immediately rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing, include under lids to remove all particles. Get immediate medical attention.
Inhalation	<ul style="list-style-type: none"> Irritation to the moist mucus membranes of the nose, throat and upper respiratory system. Pre-existing upper respiratory and lung diseases may be aggravated. 	<ul style="list-style-type: none"> Move exposed individual from the area into fresh air. Get medical attention if coughing and other symptoms do not subside.
Ingestion	<ul style="list-style-type: none"> Internal discomfort or irritation to the digestive tract. 	<ul style="list-style-type: none"> Do NOT induce vomiting. If individual is conscious, use water to rinse out the mouth and have the individual drink plenty of water. If vomiting occurs, lean patient forward or place on left side (head-down position, if possible) to maintain an open airway and prevent aspiration. Observe the individual carefully. Never give anything by mouth to an unconscious individual (i.e. showing signs of being sleepy or with reduced awareness). Get immediate medical attention.

5 – FIRE-FIGHTING MEASURES

Conditions of Flammability	Not flammable or combustible.
Suitable Extinguishing Media	Use water spray, alcohol-resistant foam, dry chemical or carbon dioxide.
Unsuitable Extinguishing Media	There is no restriction on the type of extinguisher.
Specific Hazards Arising From the Product	None known.
Special Protective Actions Equipment and Precautions for Fire-Fighters	<ul style="list-style-type: none"> Wear self-contained breathing apparatus and protective gloves for firefighting. Equipment should be thoroughly decontaminated after use. Prevent spillage from entering drains or water courses.

6 – ACCIDENTAL RELEASE MEASURES


Personal Precautions, Protective Equipment and Emergency Procedures	<ul style="list-style-type: none"> Wear appropriate personal protective equipment to prevent contamination of skin, eyes and personal clothing (see Section 8). Avoid contact with the product: <ul style="list-style-type: none"> Avoid contact to the product, freshly cured product or prolonged wet skin exposure to the product. Avoid actions that cause dried product to become airborne. Avoid inhalation of dried product dust. Use a scoop or moist paper towel(s) to clean up spilled product.
Environmental Precautions	<ul style="list-style-type: none"> Prevent further spillage. Keep product away from drains, surface and ground water.
Methods and Materials for Containment and Cleaning Up	<ul style="list-style-type: none"> Use a scoop or moist paper towel(s) to clean up spilled product. Dispose of in a suitable sealed container. Dispose of waste product according to local state and federal regulations. Cover drains. Wash spilled area with large amounts of water and dry thoroughly.

7 – HANDLING AND STORAGE

Precautions for Safe Handling	<ul style="list-style-type: none"> Avoid contact with skin and eyes. Avoid inhalation of dried product. Wear personal protective equipment. Remove contaminated clothing and personal protective equipment before entering eating areas. Wash hands after handling product. Remove contaminated clothing and personal protective equipment. Wash before reuse. Avoid contact with ammonia, ammonia nitrate, chlorine or chlorine containing items.
Conditions for Safe Storage	<ul style="list-style-type: none"> Air and moisture sensitive. Store at room temperature in a dry area. Store tightly closed in the original packaging. Keep dry until used. Protect from excessive heat and cold. Protect packaging against physical damage. Store away from incompatible materials, and from foodstuffs and beverages. Do not store near ammonia, ammonia nitrate, chlorine or chlorine containing items.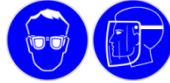

8 – EXPOSURE CONTROLS / PERSONAL PROTECTION

Control Parameters	The product does not contain any relevant quantities of materials with critical values that have to be monitored at the workplace.	
Appropriate Engineering Controls	Use and store in a well-ventilated area.	
Individual Protections Measures, Such as Personal Protective Equipment (PPE)	Eye / Face Protection 	<ul style="list-style-type: none"> Safety glasses with side shields. Tightly sealed safety goggles. Face shields. Contact lenses should not be worn.
	Skin Protection 	<ul style="list-style-type: none"> Glove material has to be impermeable and resistant to the product: <ul style="list-style-type: none"> Exam gloves: nitrile rubber or latex. Recommended thickness of the glove material: $\geq 0.38\text{mm}$ Use proper glove removal technique (without touching glove's outer surface) to avoid skin contact with this product. Dispose of contaminated gloves after use in accordance with applicable laws and good laboratory practice. Wash and dry hands after handling product.
		<ul style="list-style-type: none"> Lightweight impermeable protective work clothing (lab coat), as appropriate, to prevent skin contact. Remove contaminated clothing and personal protective equipment before entering eating areas. Remove contaminated clothing and personal protective equipment. Wash before reuse.
	Respiratory Protection 	<ul style="list-style-type: none"> Surgical face mask Handle product in a well-ventilated area.
	Thermal Hazards	Not applicable
	Hygiene Measures	<ul style="list-style-type: none"> Handle in accordance with good industrial hygiene and safety practice. Keep away from foodstuffs and beverages. Wash hands before breaks and at the end of work.

9 – PHYSICAL AND CHEMICAL PROPERTIES

Appearance	State	Solid (thick paste / putty)	Upper / Lower Flammability or Explosive Limits	Not available
	Colour	White		
Odour	No distinct odour		Vapour Pressure	Not available
Odour Threshold	Not available		Vapour Density	Not available
pH	>12 (Basic)		Relative Density	Not available
Melting Point / Freezing Point	Not available		Solubility (with water)	Slightly soluble
Initial Boiling Point and Boiling Range	Not available		Partition Coefficient: n-octanol / water	Not available
Flash Point	Not available		Auto-ignition Temperature	Not available
Evaporation Rate	Not available		Decomposition Temperature	Not available
Flammability	Not available		Viscosity	Not available

10 – STABILITY AND REACTIVITY

Reactivity	Contact with water will result in hydration of the product and produces (caustic) calcium hydroxide.
Chemical Stability	<ul style="list-style-type: none"> Stable under recommended storage conditions. Keep dry.
Possibility of Hazardous Reactions	Not available
Conditions To Avoid	Avoid unintentional contact with water to prevent premature setting/curing; contact will result in hydration of the product and produces (caustic) calcium hydroxide.
Incompatible Materials	Incompatible with acids, ammonia, ammonia nitrate, ammonium salts, aluminum metal and chlorine.
Hazardous Decomposition Products	<ul style="list-style-type: none"> Decomposition will not spontaneously occur. Contact with water will result in hydration of the product and produces (caustic) calcium hydroxide.

11 – TOXICOLOGICAL INFORMATION

Emergency Overview	This product is a white paste that poses little immediate hazard. A single-term exposure to the paste is not likely to cause serious harm. However, exposure of sufficient amount and duration (either single or multiple) to the moist product can cause serious irritation to the skin and eyes in the form of chemical (caustic) burns. Similar tissue damage can occur if wet or moist areas of the body are exposed for sufficient duration to the product.
---------------------------	--

Numerical Measures of Toxicity

Chemical Name	CAS #	ACGIH	OSHA PEL	LD ₅₀
Tricalcium silicate	12168-85-3	10 mg/m ³ total dust	5 mg/m ³ respiratory dust 10 mg/m ³ total dust	Not available
Dicalcium silicate	10034-77-2			
Zirconium oxide	1314-23-4	5 mg/m ³ total dust	5 mg/m ³ TWA	Not available
Tantalum pentoxide	1314-61-0	5 mg/m ³ total dust	5 mg/m ³ total dust	Oral: >4000 mg/kg Rat: 8000 mg/kg
Calcium sulfate	10034-76-1	10 mg/m ³ (TWA) inhalable fraction	15 mg/m ³ total dust 5 mg/m ³ respiratory fraction	No LD50 / LC50

Routes of Exposure	Symptoms Related to Physical, Chemical and Toxicological Characteristics	Delayed and Immediate Effects Chronic Effects from Short and Long-Term Exposure	Target Organ(s)
Skin Contact	Skin irritation, drying, cracking or fissuring of the skin.	Severe skin damage in the form of chemical (caustic) burns.	Skin
Eye Contact	Severe irritation and/or inflammation.	Blindness in severe cases.	Eye
Inhalation	Irritation to the moist mucus membranes of the nose, throat and upper respiratory system.	Pre-existing upper respiratory and lung diseases may be aggravated.	Mucous membranes of nose, throat and upper respiratory system.
Ingestion	Internal discomfort or irritation to the digestive tract.	Ill effects are possible if large quantities are consumed.	Digestive system

Acute Toxicity	Product is not classified as a toxic substance.
Skin Corrosion / Irritation	May cause skin irritation.
Serious Eye Damage / Irritation	May cause serious eye irritation.
Respiratory or Skin Sensitization	No allergic potential after multiple uses and good tolerance by subcutaneous tissues.
Germ Cell Mutagenicity	No mutagenic effect.
Carcinogenicity (NTP, IARC, OSHA)	Ingredients not listed as carcinogens.
Reproductive Toxicity	Not available
Specific Target Organ Toxicity (STOT) – Single Exposure	Product is not classified as a toxic substance.
Specific Target Organ Toxicity (STOT) – Repeated Exposure	Product is not classified as a toxic substance.
Aspiration Hazard	Not available
Interactive Effects	Not available
Other Information	Not applicable

12 – ECOLOGICAL INFORMATION

Toxicity	Not available	Mobility in Soil	Not available
Persistence and Degradability	Not available	Other Adverse Effects	No known significant effects or critical hazards.
Bioaccumulative Potential	Not available		

13 – DISPOSAL CONSIDERATIONS

Disposal Methods	<ul style="list-style-type: none"> For the safe handling of disposing of this product, wear appropriate personal protective equipment to prevent contamination of skin, eyes and personal clothing (see Section 8). Dispose of waste product or contaminated product in a suitable sealed container. Dispose of product according to local state and federal regulations.
Physical / Chemical Properties Affecting Disposal Options	No known precautions.
Discourage Sewage Disposal	Do not dispose of product down drains or from entering sewage systems.
Special Precautions for Incineration or Landfill	No known precautions.

14 – TRANSPORT INFORMATION

UN Number	Not applicable, the product is not classified as Dangerous Goods.
UN Proper Shipping Name	Not applicable, the product is not classified as Dangerous Goods.
Transport Hazard Class(es)	Not classified as a transport hazard in accordance with the UN Model Regulations.
Packing Group	Not applicable, the product is not classified as Dangerous Goods.
Environmental Hazards	<ul style="list-style-type: none"> Product is not known as a marine pollutant according to the IMDG Code. Product is not an environmental hazard according to the UN Model Regulations, ADR, RID and ADN.
Precautions for User	Not applicable
Transport in Bulk (According to Annex II of MARPOL 73/78⁹ and the IBC Code)	Not applicable

15 – REGULATORY INFORMATION

Status under REGULATION (EC) No 1272/2008 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 16 December 2008 on classification, labelling and packaging of substances and mixtures, amending and repealing Directives 67/548/EEC and 1999/45/EC, and amending Regulation (EC) No 1907/2006;	<ul style="list-style-type: none"> This device is classified as an irritant: <ul style="list-style-type: none"> Skin irritation (Category 2) Eye irritation (Category 2)
Status under USDOL-OSHA Hazard Communication Rule, 29 CFR 1910.1200:	<ul style="list-style-type: none"> The ingredients in this product (calcium silicate compounds, tantalum pentoxide, zirconium oxide and calcium sulfate) are considered to be “hazardous chemicals” under this regulation.

16 – OTHER INFORMATION

Date of Preparation	2023-08-15
----------------------------	------------

This product should be used according to manufacturer's directions. Safety precautions presented above may not be adequate for all individuals or situations. Inexperienced product users should obtain proper training before using or handling this product. Users have the responsibility to comply with all laws and procedures applicable to the products' safe handling and use, determine the suitability of the product for its intended use, and understand possible hazards associated with mixing this product with other materials. The above information is believed to be correct but does not purport to be all inclusive and shall be used only as a guide. The information in this document is based on the present state of our knowledge and is applicable to the product with regard to appropriate safety precautions. It does not represent any guarantee of the properties of the product. The supplier and/or manufacturer of this product shall not be held liable for damage resulting from handling, misuse or from contact with this product.